



## SCALP TREATMENT PROTOCOL

Before beginning this treatment protocol, please read the “Getting Started” documents and familiarize yourself with the InterX 5002 and the treatment methods. If you have any questions about these instructions, some may be answered in the operations manual that comes with the device, or contact your sales representative.

### Scan and Target


Use the comb attachment electrode for this application. If you are bald, you may use the built-in electrode, or dome accessory.



Turn the device on and select the desired setting and intensity.

For the first 1-2 treatments, use 60hz then 180hz. Use 240hz on third treatment then rotate.

If you are bald, lightly dampen the skin to facilitate stimulation.

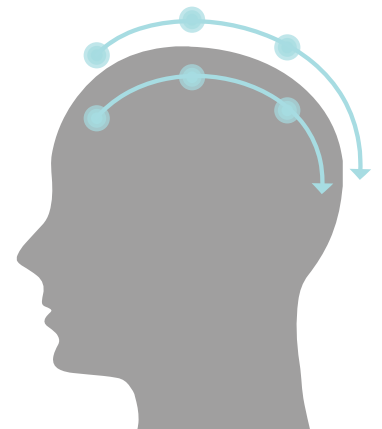

# 1.

Scalp treatment helps reduce neuropathic pain. If the pain originates on the right side of the body, for example, in the right hand, you can treat the left side of the head in the scalp. In other words, the right side of the brain controls the left side of the body and vice versa.

# 2.

Areas in the scalp that are sharp or more tender must be treated more. This is the area of the brain most affected. Some of the worst pain is reduced by treatment in the scalp using the comb electrode. Suggested time of treatment is 10-15 minutes.

# 3.

Make sure you maintain good contact between the comb electrode and the skin on the scalp when treating over the scalp, covering the whole scalp in columns and moving slowly.



Place the electrode onto the area around the anterior hairline to set the intensity. This ensures that you have good skin contact for the initial sensation to be correct.


Comb the electrode very slowly through the patient's hair two or three times from the anterior hairline to the posterior hairline.

## Target


Target those points where you feel stronger stimulation. Target any sensitive points for 1 minute each. You may need to reduce the intensity while treating these points.

Continue like this until the whole scalp has been scanned and active points targeted.

You may have a delayed sensation, in which case it will be necessary to hold the comb in one spot for 10 seconds before the stimulation penetrates the scalp. This delayed response will shorten with subsequent treatments.

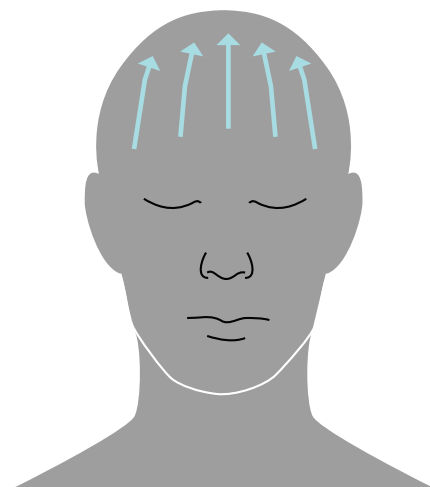

Anterior hairline

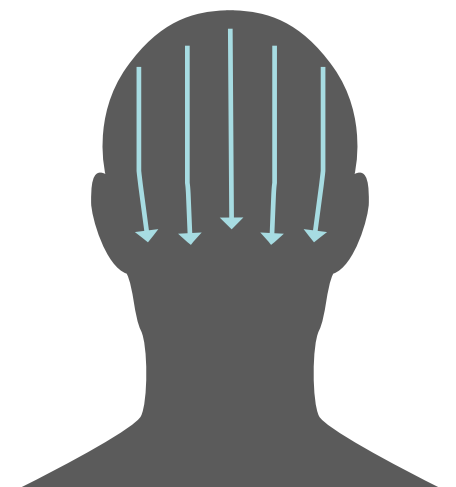

Posterior Hairline